QuikRead® iFOB
Sampling Set

IVD

REF

151052

CONT

50 x 2 ml

Sampling Instructions

Do not drink the liquid in the sampling tube.

Take care not to bend or break the tip of the tube.

- 1 Obtain a sample of faeces in a clean container or on clean collecting paper. Avoid contact with the toilet water when you are collecting the sample. If there is visible blood in the faeces sample or in case of diarrhea, do not collect a sample but wait for the next bowel movement and collect a sample from that.
- 2 Unscrew the top of the sampling tube containing the sample collecting stick. Collect the sample by poking the sample collecting stick (attached to the tube cap) in three (3) different places on the faeces sample.
- 3 Insert the sample collecting stick back into the sampling tube and screw the cap on firmly.
- 4 Write your name and the date on a separate label and attach the label to the sampling tube.
- 5 Return the iFOB sampling tube to the laboratory or doctor as soon as possible after sampling. Protect the sampling tube from breakage or leakage during transportation.
- 6 The sample in the sampling tube can be stored in a refrigerator (2...8°C) for up to 7 days or at room temperature (max. 25°C) for up to 5 days. Storage in refrigerator is however preferred.

Consult the separate drawing containing patient instructions.

Storage

Store the sampling tubes at 2...25°C.

Warnings and precautions

- Do not use the sampling tubes after the expiry time marked on the label as yyyy-mm.
- The reagent contains 0.1% formaldehyde. May cause cancer (Carc. 1B, H350) and may produce an allergic reaction (EUH208). Obtain special instructions before use (P201). Wear protective gloves/protective clothing (P280). IF ON SKIN: Wash with plenty of soap and water (P302+P352). Dispose of contents according to national and local law (P501).
- Reagents contain < 0.1% sodium azide, which is not considered a harmful concentration. Sodium azide liberates toxic gas when in contact with acids. Azides may react with metal plumbing, forming explosive compounds. Azide build-up can be avoided by flushing with a large volume of water when disposing of the reagents.
- All patient samples should be handled and disposed of as potentially infectious material.
- When used in accordance with Good Laboratory Practice, good occupational hygiene and the instructions for use, reagents supplied should not present a hazard to health.

Danger

Consult instructions for use:
QuikRead go iFOBT, Ref. 151051

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QuikRead go® is a registered trademark of Aidian Oy.

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