

Crown Name(WH) United Co., Ltd. 19F Bund House 7 Building, No.8 Sanyang Road, Wuhan 430010, China

Tel: 86-27-82280908-8818 Email: sales3@crownname.com

Date: May 24, 2024


Product Technical Sheet

Product Description

Product: SMS examination skirt

Item No.: 481689 Colour: navy blue Material: SMS Thickness: 45g

Size: Please see below table

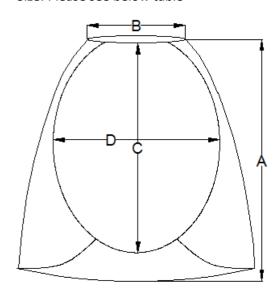

Size Chart(cm)		
		Tolerance
Α	68	±1
В	65	±1
С	68	±1
D	23	±1

Packing: 10pcs/bag, 50pcs/ctn
Carton measurement: 52*30*26cm

Suggested Sterilization

ETO: (pre-treatment cycle 16-72 hrs) (gas contact – 3 hrs/850 mg) (evaporation 48 hrs at 40°C)

Suggested Storage:

Temperature: between -5°C and 40°C

Humidity : below 80%

Sunlight : not to expose to direct sunlight

Shelf-Life : 5 years

Intended Use: It is non-woven material and standard fabric for non-toxic workplace. It can be used in hospital and pharmaceutical applications.

Applicable Directive/Regulation:

MDR for medical device Class I EN 13795:2011+A1:2013



Crown Name(WH) United Co., Ltd. 19F Bund House 7 Building, No.8 Sanyang Road, Wuhan 430010, China

Tel: 86-27-82280908-8818 Email: sales3@crownname.com

Applicable Standards:

ISO9001

ISO13485

EN13795

Risk Analysis

The raw material is approved to be safe to human health (refer to the manufacturer's safety data sheet). As for the product, no hazard is foreseeable to the safety and health of user.

Instruction for Use:

Usage:

- 1. Take it out from the package (box or polybag), then unfold it
- 2. Put legs into the hole of skirt, adjust elastic on the waist.

Cautions:

- 1. This product is for single use. Do not re-use it.
- 2. This product is for protection of part of body, not for overall protection.
- 3. This product is for hygienic protection, not for any other personal protection against potential risk.
- 4. This product is non-sterile. It must be sterilized before being used as sterile wear.
- 5. Choose right size according to the user's stature and figure.