

湖北鸿瑞塑料有限公司
HUBEI HONGRUI PLASTICS CO., LTD.
6 Street of Huangjing, Xiantao, Hubei, China

Oct 15, 2019


Product Data Sheet

Customer : Norengros AS, Fyrstikkalléen 3A, 0661 Oslo, Norway

Product Description

Product : CPE Shoe Covers (Art. no. 81031)
Colour : Blue
Material : 95% LDPE, 5% blue masterbatch
Thickness : 30 micron (before embossing), 150-160 micron (with embossment)
Weight : net 3.5 grams (+/-5%)
Size : Length 450 mm (+/- 3%) Height 150 mm (+/- 3%)
Natural openingf: about 110mm
Stretch opening: 450mm (+/-3%)

Elastic band : 1.1 mm x 157 mm (latex free)

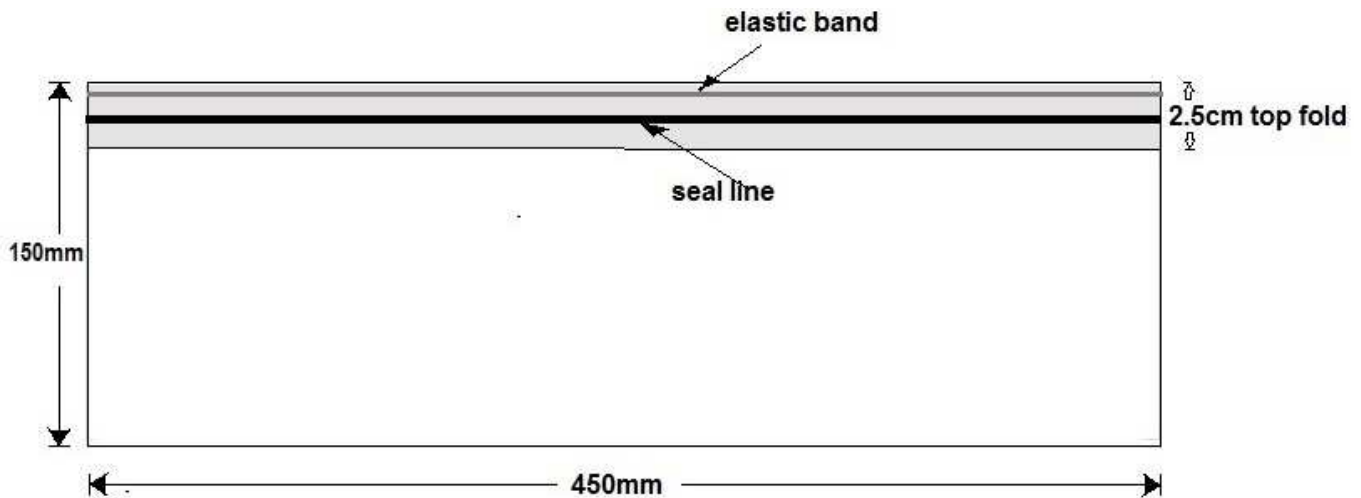

Packing : 10 pcs. per bundle, 10 bundles per polybag, 2000 pcs. per export carton

Carton meas. : 43 x 35 x 38 cm

G.W./N.W. : 7.4 kgs / 7.0 kgs

Suggested Sterilization

ETO: (pre-treatment cycle 16-72 hrs)
(gas contact – 3 hrs/850 mg)
(evaporation 48 hrs at 40°C)

Suggested Storage:

Temperature : between -5°C and 40°C
Humidity : below 80%
Sunlight : not to expose to direct sunlight
Shelf-Life : 3 years

Suggested Disposal Method:

Recycling or burning

Intended Use:

It is in liquid repellent polyethylene plastic to protect shoes from splashes or smearing. It can be used as footwear in cleanroom, hospital, or industry.

Applicable Directive/Regulation:

Directive 93/42/EEC for medical device class I
Regulation (EU) 2016/425 for PPE cat. I

Risk Analysis

The raw material is approved to be safe to human health (referring to the manufacturer's safety data sheet). As for the product, no hazard is foreseeable to the safety and health of user.

Cautions:

1. This product is for single use. Do not re-use it.
2. This product is for hygienic protection, not for any other personal protection against potential risk.
3. This product is non-sterile. It must be sterilized before being used as sterile wear.
4. Do not wear it directly without any shoes. It may cause skin discomfort when touching skin directly.
5. Do not wear it in the place where electrostatic prevention is requested. It is not anti-static treated.
6. Choose right size according to the user's foot size.

Approved by:

LI LEZHI

General Manager