

Date: Jul. 1, 2026

## Product Technical Sheet

### Product Description

Product: Sterilized Surgical gown with reinforcement, two hand towels

Item No.: 411218/406723/406724/411219

Colour: blue

Material: SMS



Thickness: 45g

Size: Please see below table

Dimension of finished product (cm)					
	411218	406723	406724	411219	
	M	L	XL	2XL	T(cm)
A	135	140	146	155	±2
B	144	154	164	170	±2
C	55	56	60	61	±1
D	29	29	29	30	±1
E	15	15	15	15	±1
wash towel	30*40				
wrap	60*60				

Packing: 1pc/bag, 50pcs/ctn

Carton meas.: 60 x 40 x 50cm

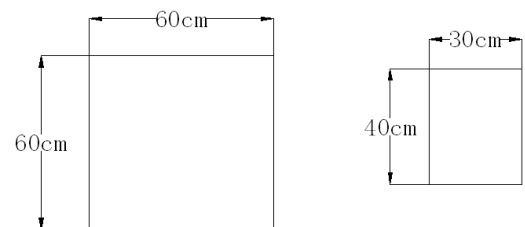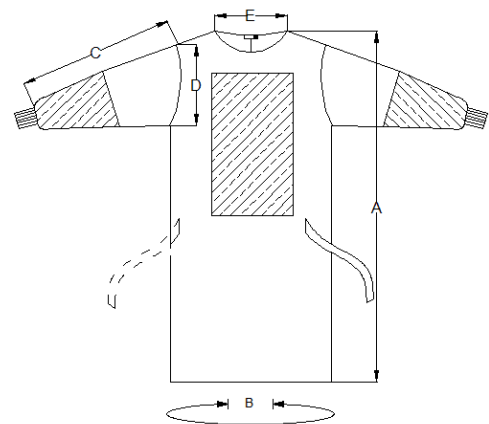

### Sterilization

EO Sterilized

### Suggested Storage:

Temperature : between -5°C and 40°C

Humidity : below 80%

Sunlight : not to expose to direct sunlight

Shelf-Life : 5 years

**Intended Use:** Used to minimize the spread of infective agents to and from patients' operating wounds, thereby helping to prevent post-operative wound infections. The use of surgical gowns with resistance to the penetration of liquids can also diminish the risk to the operating staff from infective agents carried in blood or bodyfluids.

### Applicable Directive/Regulation:

Medical Device Regulation (EU) 2017/745 Class I sterile

### **Applicable Standards:**

ISO 13485:2016

EN 13795-1:2019

### **Risk Analysis**

The raw material is approved to be safe to human health (refer to the manufacturer's safety data sheet). As for the product, no hazard is foreseeable to the safety and health of user.

### **Instruction for Use:**

#### **Usage:**

1. Perform thorough hand hygiene before handling the sterile gown.
2. Grasp the exposed inner side of the sterile gown, lift it away from the sterile surface, and fully unfold it without touching the outer sterile area.
3. Insert both hands into the armholes, then slide your forearms fully into the sleeves — do not extend your hands through the cuffs if you are using the closed gloving method.
4. An assistant will fasten the neck ties and the inner waist tie for you, only touching the inner side of the gown.
5. Complete closed gloving first, then grasp the sterile outer tie, pass the disposable tab to the assistant, turn 3/4 circle, and take back the tie to fasten it at your waist.
6. Keep your hands above your waist and in front of you at all times, avoid touching non-sterile items including your face, mask and hat.

#### **Cautions:**

1. This product is for single use. Do not re-use it.
2. This product is for protection of part of body, not for overall protection.
3. This product is for hygienic protection, not for any other personal protection against potential risk.
4. Do not wear it directly without any clothes. It may cause skin uncomfortably when touching skin directly.
5. Do not wear it in the place where electrostatic prevention is requested. It is not anti-static treated.
6. Choose right size according to the user's stature and figure