

Date: Feb. 28, 2025

## **Product Technical Sheet**

### **Product Description**

Product: Nurse gown, velcro on neck, ties on waist, knitted cuffs


Item No.: 464398/464399/464400

Colour: Blue

Material: SMS

Thickness: 35gsm

Size: Please see below table

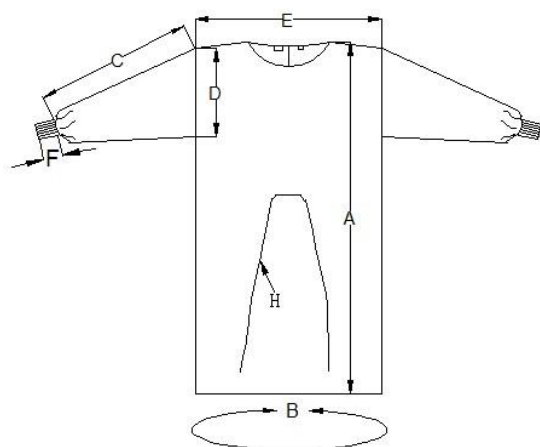

| Dimension of finished product(cm) |     |     |     |     |       |
|-----------------------------------|-----|-----|-----|-----|-------|
|                                   | S   | M   | L   | XL  | T(cm) |
| A                                 | 110 | 115 | 135 | 145 | ±2    |
| B                                 | 130 | 135 | 155 | 172 | ±2    |
| C                                 | 59  | 59  | 62  | 69  | ±2    |
| D                                 | 26  | 26  | 30  | 32  | ±1    |
| E                                 | 63  | 66  | 75  | 84  | ±2    |
| F                                 | 6   | 6   | 6   | 6   | ±1    |
| G                                 | 180 | 200 | 200 | 200 | ±5    |
| H                                 | 35  | 35  | 35  | 35  | ±3    |

Packing: 10pcs/bag, 100pcs/ctn

Carton meas. : S/M 52\*34\*35cm, L 59\*35\*38cm, XL 59\*39\*43cm

### **Suggested Sterilization**

ETO: (pre-treatment cycle 16-72 hrs)

(gas contact – 3 hrs/850 mg)

(evaporation 48 hrs at 40°C)

### **Suggested Storage:**

Temperature : between -5°C and 40°C

Humidity : below 80%

Sunlight : not to expose to direct sunlight

Shelf-Life : 5 years

**Intended Use:** Used for general isolation in medical institutions' clinics, wards, laboratory, etc.

### **Applicable Directive/Regulation:**

Medical Device Regulation (EU) 2017/745

### **Applicable Standards:**

ISO9001

ISO13485:2016

### **Risk Analysis**

The raw material is approved to be safe to human health (refer to the manufacturer's safety data sheet).  
As for the product, no hazard is foreseeable to the safety and health of user.

### **Instruction for Use:**

#### **Usage :**

1. Take it out from the package (box or polybag), then unfold it
2. Put arm into the hole of sleeve, tie up the band at the back of neck and waist.

#### **Cautions:**

1. This product is for single use. Do not re-use it.
2. This product is for protection of part of body, not for overall protection.
3. This product is for hygienic protection, not for any other personal protection against potential risk.
4. This product is non-sterile. It must be sterilized before being used as sterile wear.
5. Do not wear it directly without any clothes. It may cause skin uncomfortably when touching skin directly.
6. Do not wear it in the place where electrostatic prevention is requested. It is not anti-static treated.
7. Choose right size according to the user's stature and figure.