

Date: Jul. 15, 2025

## Product Technical Sheet

### Product Description

Product: SMS colonoscopy short pants


Item No.: 481691/481690/481692

Color: navy blue

Material: SMS

Thickness: 45g

Size: Please see below table

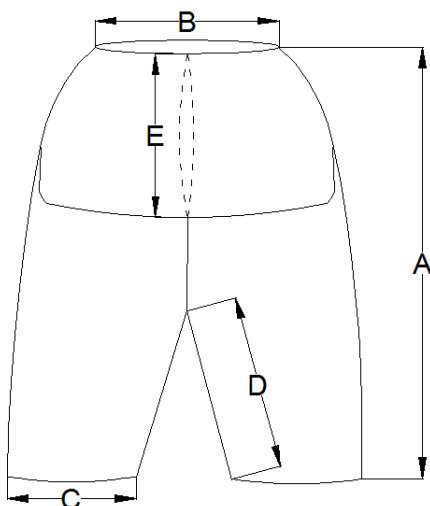

Size Chart(cm)				
	M 481691	L 481690	XL 481692	Tolerance
A	55	63	72	±1
B	60	62	66	±1
C	28	28	28	±1
D	23	30	39	±1
E	23	23	23	±1

Packing: 10pcs/bag, 50pcs/ctn

Carton measurement: 48\*28\*30cm

### Suggested Sterilization

ETO: (pre-treatment cycle 16-72 hrs) (gas contact – 3 hrs/850 mg) (evaporation 48 hrs at 40°C)

### Suggested Storage:

Temperature : between -5°C and 40°C

Humidity : below 80%

Sunlight : not to expose to direct sunlight

Shelf-Life : 5 years

**Intended Use:** It is non-woven material and standard fabric for non-toxic workplace. It can be used in hospital and pharmaceutical applications.

### Applicable Directive/Regulation:

MDR for medical device Class I

EN 13795:2011+A1:2013



Crown Name(WH) United Co., Ltd.

Bund House 7 Building 19F, No.8 Sanyang Road, Wuhan 430010, China

Tel: 86-27-82280908-8818 Email: sales3@crowname.com

---

**Applicable Standards:**

ISO 9001

ISO 13485

EN 13795

**Risk Analysis**

The raw material is approved to be safe to human health (refer to the manufacturer's safety data sheet).

As for the product, no hazard is foreseeable to the safety and health of user.

**Instruction for Use:**

**Usage:**

1. Take it out from the package (box or polybag), then unfold it
2. Put legs into the pants, adjust elastic on the waist.

**Cautions:**

1. This product is for single use. Do not re-use it.
2. This product is for protection of part of body, not for overall protection.
3. This product is for hygienic protection, not for any other personal protection against potential risk.
4. This product is non-sterile. It must be sterilized before being used as sterile wear.
5. Choose right size according to the user's stature and figure.